


OmniView Quick Sheet

Admissions and Patient Setup

To admit a new patient into the OmniView System use the following steps:

1. Hook your patient up to their bedside monitor
2. Click the  icon that corresponds with the bedside monitor that your patient is connected.

3. Click **Patient Information Setup**

Set information for the patient's ID, name, age, sex, type, blood, room, doctor and diagnose notes

4. Click on **Parameter Information Setup**

type to change the displayed parameter on the main board. Totally HR, ST, PR, SPO2, C.O., ETCO2, IMCO2, AWRR, NIBP, IBP1, IBP2 and TEMP can be chosen

5. **Waveform Information Setup**

Select waveforms in the main monitoring interface and view bed. The main monitoring interface can display 2 channel waveforms.


There are 11 selectable waveforms: ECG (I, II, III, aVR, aVL, aVF, V), PLETH, IBP1, IBP2 and ETCO2.

5. Click **Monitor Function Setup**

- ECG Lead: Select HR calculating source.
- ECG Gain: Select ECG gain mode.
- ECG Filter: Select ECG filter mode.
- ECG Notch: Select ECG filter notch mode.
- Pacemaker Detection: Enable/disable pacemaker detection.
- Inflation Mode: Select NIBP inflation mode.
- Time Interval(mins): Select inflation(auto mode) time interval.

6. Finally Click "Exit"

Discharge Patient

1. Click the  icon on the Patient you wish to discharge
2. Click the "New Patient" button to clear our your discharged patient's information

Main Function Icon Area

These icons are located on the Right Side of the Screen. There are 7 Icons.



Help Icon:

Displays the Help Information about System Operations.



Alarm History Review:

The History Review can display 5000 items of parameter alarms for review.



Printer:

This Icon allows access to printer functionality.



History Data Review:

Review of patient history including Trending Graphs and Full Disclosure of ECG waveforms.



System Setup:

Allows the custom configuration of how the central station will function.



Alarm Sound Switch:

“Click” to turn Alarm Sounds On or Off.




Exit Button:





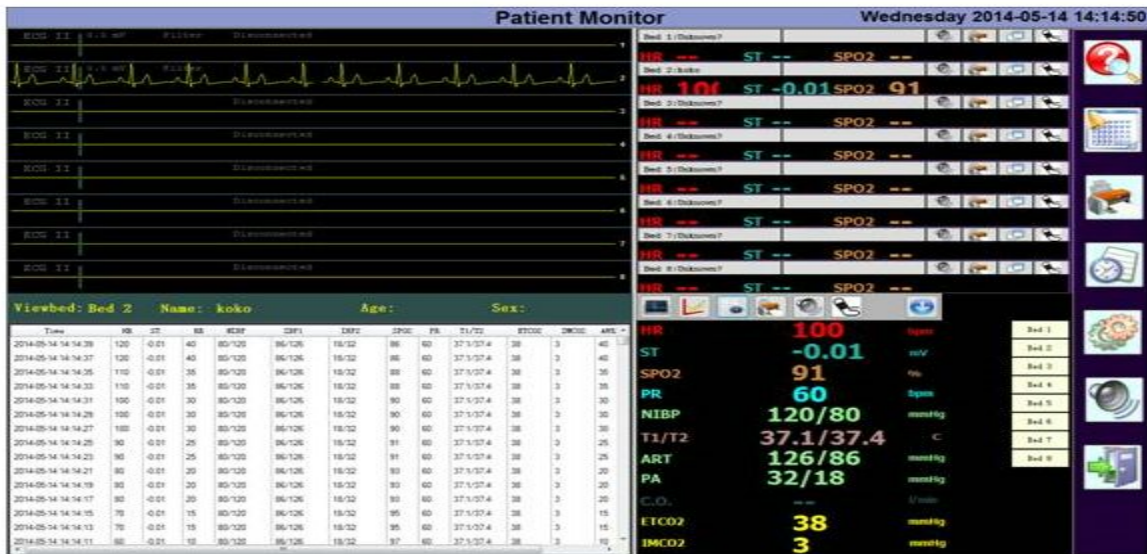


Use this button to exit out of the Omniview program and return to the Windows Desktop.

Individual Patient Review

- To bring up the individual patient information screen. Double click on the desired patient's name to bring up the following screen.



Individual Patient Information Icon Explanation

- () **Waveform icon:** Click to display the viewbed's 5 waveforms.
- () **Trend graph icon:** Click to display the viewbed's trend graph (see the details in the relevant chapter).
- () **Trend table icon:** Click to display the viewbed's trend table (see the details in the relevant chapter).
- () **Print icon:** Click to print the viewbed relevant current screen display data, including trend graph, trend table and waveforms.
- () **Alarm sound switch icon:** Click to set the alarm sound to be ON or OFF.
- () **NIBP test icon:** Click to start/stop NIBP test.
- () **Exit icon:** Exit from viewbed monitoring window and enter the main monitoring interface.

- When you select the "Trend Graph Icon" or "Waveform Icon" or "Trend Table Icon" click the Print icon to print off the currently selected screen

Reports

Current Patient Vitals

1. Click the “Printer Icon” on the individual patient work bar
2. Click “Quick Print”
3. Press the ENTER button on the keyboard or click the “OK” in the window

Previous Patient History/Alarms

1. Click on “History Patient Data” icon
2. To select your previous patient you wish to recall, click “Data Read”
3. Select your previous patient then click “Read”
4. Now that you have recalled the patient information,
 - ❖ To view a specific Vital Sign/ECG Lead
 - Click on the drop box in the lower center right corner then select desired vital/lead
 - ❖ How you view your Patient's Vital Sign/ECG Lead,
 1. Click on the Drop Box in the lower center left
 2. Select Either:
 - “**Trend Graph**” - Patient's numerical vital line graphed over their stay
 - “**Waveform**” – Patient`s waveform over their stay, Double Click on waveform to expand then click “Print”, “Quick Print”
 - “**Trend Table**” – Patient vital signs in numerical table format, to condense their vital signs instances, select a larger time interval

Current Patient Alarm History Review

1. Click on the “Alarm History Review” icon on the main screen
2. Select which current Patient you by Clicking on the drop bar in the lower left corner
3. Click on the “Print” icon then “Quick Print” for a listing of their alarms.